

Internet Explorer

- 1) Click on the link for Genie 2015.2 English/French to begin the download:



Software Support




Latest Software Releases

Do you have the most recent software releases on your computer?

[Genie 2015.1 English/French](#) (USB 1.4 GB)

[Genie Medical 2013.1](#) (535 MB)
12.5 Mb - English only


- 2) Select "save" from the window that appears:

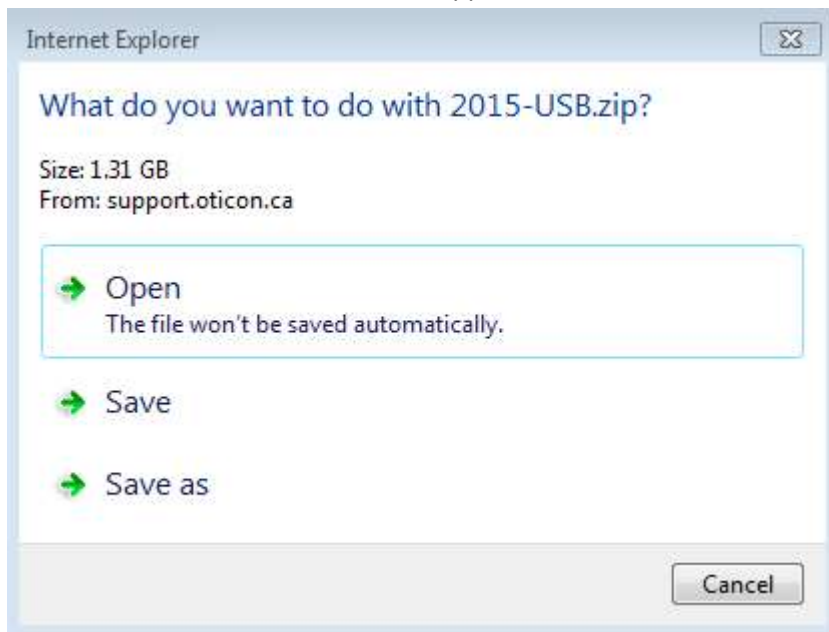

- 3) When the download is complete, click on “View Downloads”:




- 4) Then Open the Genie2015-USB.zip file:

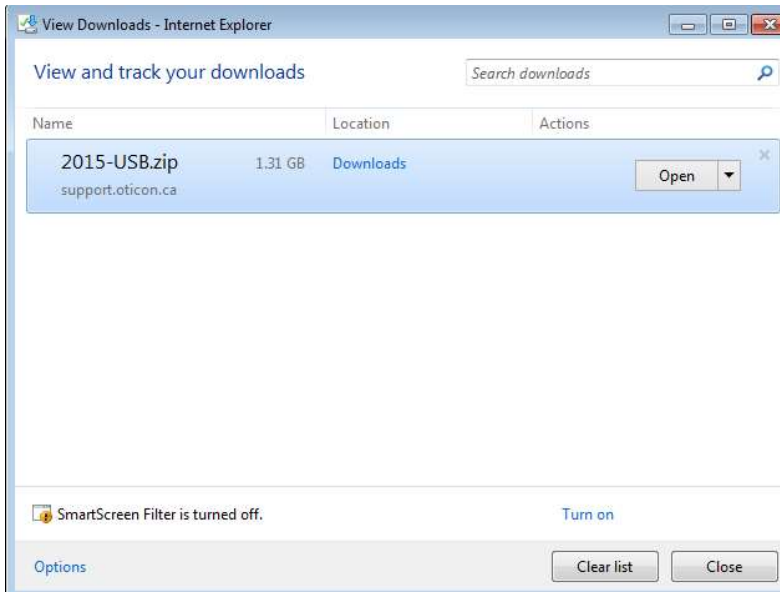


- 5) Create a new folder called “Genie 2015.2” on your desktop




- 6) Right click on the Genie 2015-USB file and select “extract all”:

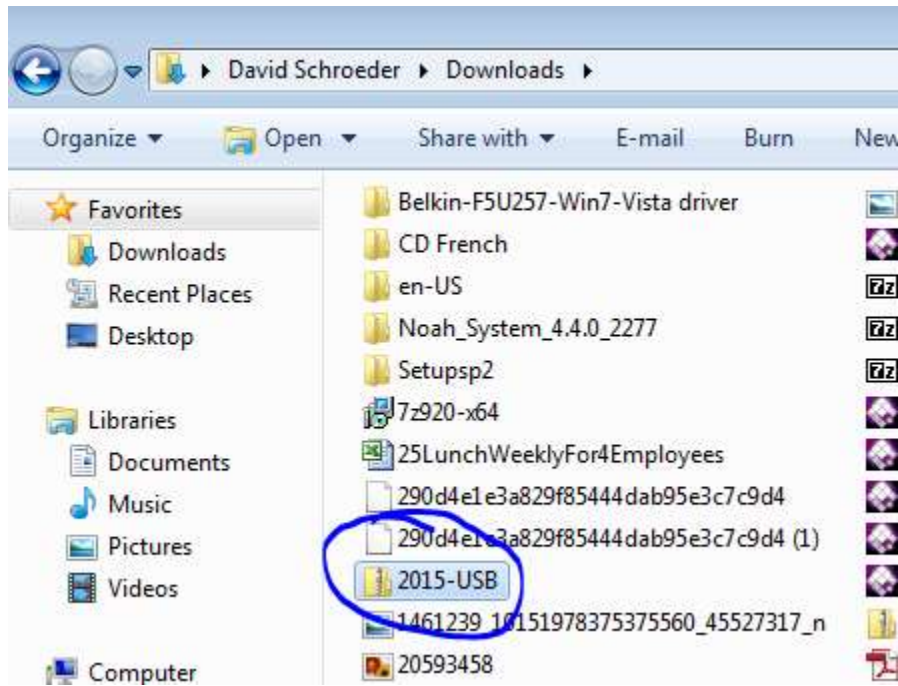

- 7) Select the folder you wish to extract the files to (please use the folder you created in Step 5):

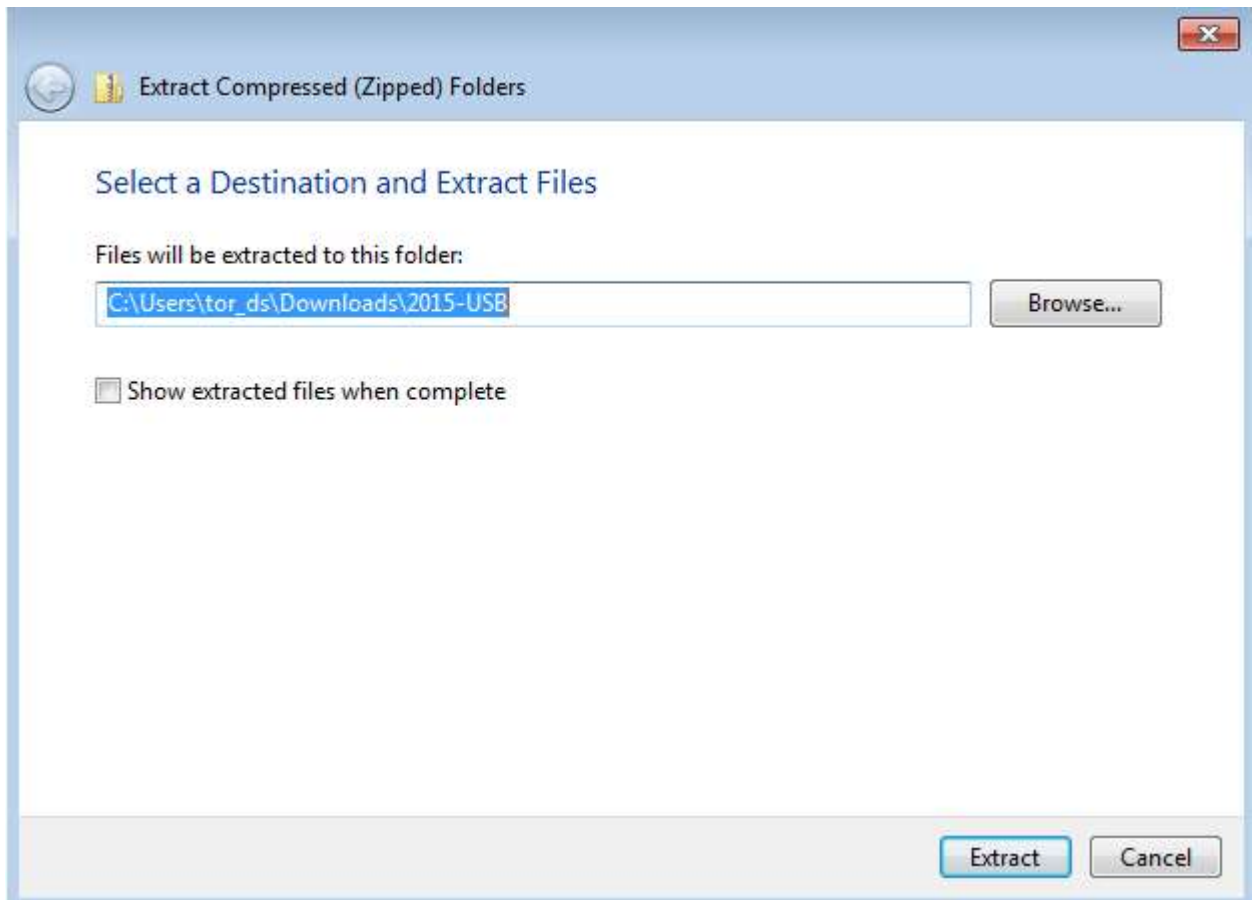

- 8) When the extraction is complete, locate the “Genie 2015.2” folder on the desktop and double click on the folder:

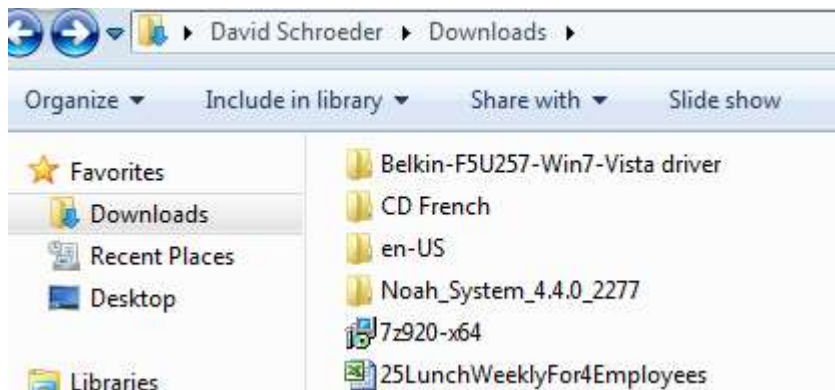

- 9) Double click the “Setup.exe” file to begin the installation process:

| | | | |
|-----------------------|--------------------|-----------------------|-----------|
| Install.exe | 08/01/2014 9:29 AM | XML Configuratio... | 2 KB |
| InstallationTools.dll | 08/01/2014 9:29 AM | Application extens... | 31 KB |
| JobSpecification | 08/14/2014 5:02 PM | XML Document | 2 KB |
| Setup | 05/27/2014 4:22 PM | Application | 1,851 KB |
| StreamerSettingsSetup | 08/14/2014 4:37 PM | Application | 21,044 KB |